	Montgomery County Fire & Rescue	Drill: 20-05
	Monthly Basic Training MBT	Month: May
	Stryker Power-Pro XT Cot	Time Required: 120 min

PREPARATION:

EQUIPMENT: Web enabled device suitable for viewing videos and access to an EMS Unit with a Stryker Power-Pro XT System (Power-Load and Power-Cot)

TARGET AUDIENCE: All firefighters/rescuers listed on the IECS list in Montgomery County

SKILLS ASSESSED: A firefighter’s ability to operate and troubleshoot the NEW Stryker Power-Pro XT System (Power-Load and Power-Cot) and safely load and unload a patient into an EMS Unit.

REFERENCES: <https://www.stryker.com/us/en/emergency-care/products/power-pro-xt.html>

OBJECTIVES: By the end of this training, personnel will be able to:


- Raise and lower the Stryker Power-Pro XT cot.
- Load and unload patients into the ambulance using the Stryker Power-Pro XT cot.
- Troubleshoot problems with the Power-Cot and Power-Load system.
- Understand how to load a Ferno cot into a Stryker Power-Load System.

BACKGROUND/SIGNIFICANCE:

It is very important that all fire/rescue members be proficient in the operation of the Stryker Power-Pro XT System (Power-Load and Power-Cot) Training in the proper operation of the cot and the safety features, reduce the chance of injuring patients or providers while operating on emergency medical incidents.

LESSON PLAN:

- VIEW MBT [Video on Stryker Power-Pro XT](#)
- VIEW Reference links above

	Montgomery County Fire & Rescue	Drill: 20-05
	Monthly Basic Training MBT	Month: May
	Stryker Power-Pro XT Cot	Time Required: 120 min

PRACTICAL SKILLS: Work with your BC to arrange access for an EMS Unit equipped with the Stryker Power-Pro XT system (Power-Load and Power-Cot).

- **In teams of two, utilizing the Power-Load and Power-Cot load and unload the cot from the ambulance.**
- **Locate and install the Mass Casualty Fastener so a Ferno Cot can be loaded/unloaded.**
- **Review when and how to change out the battery on the cot.**
- **Loading: In teams of two, practice the scenario of the Power-Load OOS, but the Power-Cot is okay**
- **Unloading: In teams of two, practice the scenario of the Power-Load okay, but the Power-Cot is OOS.**
- **Review the regular cleaning and decontamination procedures for the Stryker Power-Pro XT System (Power-Load and Power-Cot).**